

**Barwon Health's new Web Viewer  
is now ready for use.**

# bmiViewer

**Provides quick, easy access to recent  
Barwon Health images and reports via the internet.  
– on any web device.**

## bmiViewer

- **Needs only a compatible web browser** (HTML 5 compatible)
- **Works on all internet devices** with a modern web browser
- **Works on- I pads & Tablets** (also smart phones – although not designed for small screens),
- **Works on all PCs & Apple Macs** – but is web browser version dependant.
- We have found that **Google Chrome works well and is able to be installed on all devices.** (PCs using Internet Explorer need a “chrome plugin” for Internet explorer installed,..... then re-boot)

## bmiViewer has:

- Recent X-ray, CT, US, MRI images since July 2012 available (more discs to be added to allow 12 months of storage)
- Recent reports since 24<sup>st</sup> Nov 2012
- a suite of basic image manipulation tools for viewing and comparison.
- (Synapse will still provide full PACS functionality for frequent users as required)

## Getting started:

Don't "search" using Google etc for "bmiViewer" – this will send you to "Body mass index" sites !!!

Type:  
bmiviewer.com.au  
into the address bar of your web browser



- First time users may be prompted to install "chrome plug in" if using Internet Explorer
- (takes 5 – 10 mins, then re-boot - Once off requirement only, no admin rights needed)



A division of **Barwon Health**

Carestream

Vue Motion

User Name:

Password:

[Change password](#) [Forgot password](#)

Log In

Vue Motion login opens

Enter Barwon Health (synapse)  
- User name (don't use bh/ prefix)  
- Password  
- login

( email [bmi@barwonhealth.org.au](mailto:bmi@barwonhealth.org.au) for login assistance)

## For fast future access - save this login page

- as a "desktop / home page shortcut"
- or
- as a "favourite/bookmark"

Search screen opens

Type surname, or UR no.

Search for patient (2)

Free Search | Last Viewed

Search for patient: gonzales

Filter by: Date - All | Patient Location - All | Studies with - All | Status - All | Modality - All | Ref. Physician

Patient Search Results (2)

Patient Name	Patient ID	Date of Birth	Gender	Patient's Most Recent Exam
GONZALES, SPEEDY	500640	1/04/1940 (72Y 2M)	M	18/06/2012 11:25 AM   XA US RF NM MR DX CT CR   LOW_EXM   00980980
GONZALES, SPEEDY	191663			CT   CHEST   01150899

Click on your patient  
(check date of birth)



Most recent images appear

Scroll through Images using:

- Slide bar
- Mouse wheel
- Finger (touch screen)

Click to display instructions

Click to open quick reference card

Image manipulation tools are at the bottom of the screen

To exit manipulation tools Re-Click pointer



